**ČIUŽINYS PRAGULŲ PROFILAKTIKAI
MATRACIS IZGULĖJUMU PROFILAKSEI
KOMPRESSORIGA MADRATS LAMATISTE PROFŪLAKTIKAKS
BUBBLE ANTIDECUBITUS MATTRESS
МАТРАС С КОМПРЕССОРОМ ДЛЯ ПРОФИЛАКТИКИ ПРОЛЕЖНЕЙ**

02-M7

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PREFACE

Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition, We wish you much success and we hope Kid-Man product will make a positive impact to your daily living.

INDICATIONS

A set, pump and mattress, is developed for bed sore proflaxis and treatment. Device is intended to be used both in domestic and hospital environment to prevent bedridden people of developing bed sores. Using this device in inproper way may cause malfunction of device or damage to product or user.

SYMBOLOLOGY

Caution

Do not bleach

Medical Device Directive 93/42/EEC

Tumble dry withlow heat

Electrical Protection Type B

Date of manufacture

Class II Equipment (Double Insulated)

Refer to manual before use

Do not iron

The marking of electrical and electronics devices according to Directive2002/96/EC. The device accessories and the packaging have to be desposed of waste correctly at the end of the usage. Please follow Local Ordinances of Regulations for disposal.

Compatible with any dry cleaning methods


No more than 80° C wash

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur.
- Keep the packed kit away from heat sources.
- DO NOT allow children to play on or operate the pump.
- Advise with physician or therapist to determine the correct adjustment and the correct use of the device.
- DO NOT use device in other way than it is intended to be used
- UAB Kasko Group is not responsible for damage occurred due to improper use of device or unauthorized repair.
- Manufacturer has right to change the information in this user manual before prior notice.

PARTS OF THE PRODUCT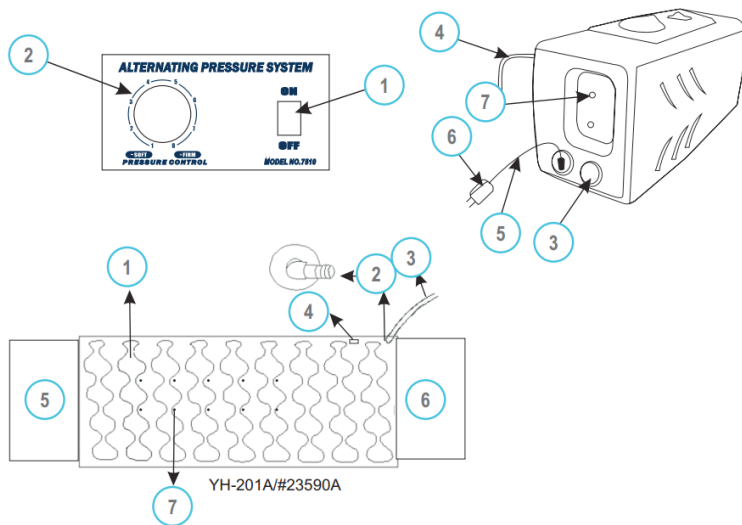


PUMP:

1. power switch
2. air pressure knob
3. fuse
4. hanging hook
5. power cord
6. plug
7. air outlet

MATTRESS:

1. air cells
2. air outlet
3. connecting tube
4. LOT number
5. head flap
6. foot flap
7. micro holes (16 pcs)

SET UP

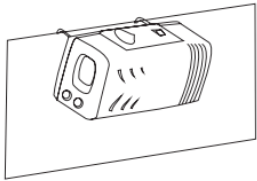
Pic. 10

Place the mattress on a common mattress, with the air hose connectors orientated to the footboard. Hang the pump onto the bed rail (footboard side), or place it on a flat surface.

Pic. 11

Connect air hose connectors from air mattress to the pump unit. Ensure the air hoses are not kinked or tucked under mattress. Plug the power cord into the electrical outlet.

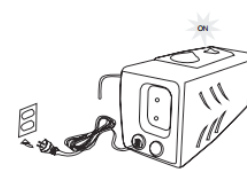Pic. 12

Turn the switch ON.

GENERAL WARNINGS

- Do not keep pump in place where it would be difficult to turn it off easily!
- Please unplug the control unit, if do not need to use for a long time!
- Avoid the cable, plug and main control unit contact with water or other fluid!
- Avoid cables damage when moved the sickbed!

- Ensure that cables are safe and tidy and not caught in the bed mechanism or trailing on the floor
- Avoid sharp objects or strong acid and alkali chemical agents from mattress.
- Avoid use the system with connecting power without any people watch
- Ensure the mattress, cover, cables and power outlet out of fire, heat or strong acid and alkali chemical agents
- Please plug the control unit into the ground power supply
- Do not modify the pump and mattress without authorization of the manufacturer and do not stuff other objects into the crack of control unit or mattress. Please don't use the parts which not belong to Kasko Group, UAB.

DO NOT USE DEVICE IF:

- The system sounds abnormal or vibrate intensively
- Control unit drop on the floor and damaged.
- Control unit drop into the water or other fluid.

USE OF THE PRODUCT

Mattress is suitable for persons up to 110 kg body weight. If overloaded, mattress do not work properly or damage to mattress or user may occur. Air pressure in the mattress should be adjusted depending on the weight of user. We suggest to use table below in order to adjust proper air pressure in the mattress.

User's weight	Position of air pressure knob
<40 kg	2-3
40 - 60 kg	3-4
60 - 70 kg	4-5
70 - 80 kg	5-6
80 - 90 kg	6-7
90 - 110 kg	8

WARNING! In case the pressure is constantly low, check for any holes (on the interchangeable elements or on the connection tubes). If it is necessary substitute the damaged connection tubes or the interchangeable elements, otherwise contact your dealer! If a rapid deflation is necessary unplug the connection tube from the mattress!

WARNING! IF MATTRESS IS NOT BEING USED FOR A LONG PERIOD, PLEASE PREPARE IT FOR STORAGE

- Switch off power from main and unplug, separate the pump with mattress and put the mattress flat.
- Fold the mattress in two from mattress foot to head
- Allow time for air to escape from mattress, and then reduce the space to storage

CLEANING AND DISINFECTION

Clean the mattress with mild soap. Do not immerse or soak pump unit. After cleaning, dry the mattress without direct sunlight exposure. Check the connection tubes are not obstructed by dust or else. In this case use a soft swab to clean the tubes.

WARNING! Turn pump off before starting cleaning procedure

WARNING! Never use acids, alkalis or solvents such as acetone

DISPOSAL

At the end of its life, the product must not be disposed of along with other domestic waste. In case of disposal don't use the inserting container for municipal waste. We recommend disposing the kit in the appropriate disposal areas for recycling.

BATTERIES (DIRECTIVE 2002/96/EC)

The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment or at retailers that provide this service. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. To remark the need to dispose of electrical equipment separately, the products is marked with crossed mobile waste bin.

TROUBLESHOOTING

If your questions can't be answered with below information, please contact your local agent directly. They might require a technician to take care of the problem.

NO LIGHTS ON UNIT	<ul style="list-style-type: none"> • Check control unit is connected to the power supply and that the unit is switched on • Check the fuse
LOW PRESSURE IN MATTRESS	<ul style="list-style-type: none"> • Check control unit is connected to the power supply and that the unit is switched on • Check mattress air supply connector is connected to the pump unit correctly • Check the connector tubes for kinks crimps or damage
CONTROL LOCK UP OR „BREEZE“	<ul style="list-style-type: none"> • Turn the control unit off and unplug the pump Wait a few minutes and plug the control unit back in. Turn on the control unit.

ELECTROMAGNETIC COMPATIBILITY DECLARATION

Guidance and manufacturers declaration action-electromagnetic emission

The device is intended for use in the electromagnetic environment specified below.
The customer or the user of the device should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations / flicker emissions IEC 61000-3-3	Compliance	

Guidance and manufacturers declaration - electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below.
The customer or the user of the device should assure that it is used in such an environment.

Immunity test	IFC 60601 test level	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 6 kV contact +/- 8 kV air	+/- 6 kV contact +/- 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	+/- 2 kV for power supply lines +/- 1 kV for input / output lines	+/- 2 kV for power supply lines Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	+/- 1 kV line(s) to line(s) +/- 2 kV line(s) to earth	+/- 1 kV differential mode Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT) 0,5 cycle 40% UT (60% dip in UT) 5 cycles 70% UT (30% dip in UT) 25 cycles <5% UT (>95% dip in UT) 5 s	<5% UT (>95% dip in UT) 0,5 cycle 40% UT (60% dip in UT) 5 cycles 70% UT (30% dip in UT) 25 cycles <5% UT (>95% dip in UT) 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptable power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	The power frequency magnetic fields should be at levels characteristic of a typical commercial or hospital environment.

NOTE UT is the a. c. mains voltage prior to application of the test level.

Guidance and manufacturers declaration – electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below.

The customer or the user of the device should assure that is used in such an environment.

Immunity test	IFC 60601 test level	Compliance level	Electromagnetic environment guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 KHz – 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz – 2,5 GHz	3 V/m	<p>Recommended separation distance: $d=1,2\sqrt{P}$ $d=1,2\sqrt{P}$ 80 MHz to 800 MHz $d=2,3\sqrt{P}$ 800 MHz to 2.5 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^A, should be less than the compliance level in each frequency range^B. Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

NOTE! At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE! These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

A Field strengths from fixed transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast can not be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeded the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorientating or relocating the device.

B Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

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• We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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