**LAUKO VAIKŠTYNĖS SU 4 RATUKAIS
ĀRA RATIŅI AR 4 RITĒNIEM
RULAATOR
4 WHEEL ROLLATOR
РОЛЛАТОР
ROLATOR CU 4 ROTI
ČTYŘKOLOVÉ CHODÍTKO**

Naudojimo ir priežiūros instrukcija
Lietošanas un kopšanas instrukcija
Kasutusjuhend
User manual
Руководство по эксплуатации
Manual de utilizare
Uživatelský manuál

04-4200F

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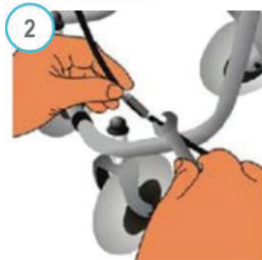
- Insert the tube of the adjustable handgrip into the frame at least to the "Maximum" marking. Fix it in the desired height with the fixing lever by turning it clockwise. Adjust both handgrips on the same height / level. Handgrips must be adjusted parallel to the walking direction! Fixing lever itself can be adjusted in the desired position by simultaneously pulling and twisting. For readjustment of the height of the handgrips unlock the fixing lever by turning it counterclockwise. Then fix the handgrip how described above.

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- The effect of the brakes can be affected by abrasion of the wheels. Therefore test regularly the brakes. For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture 1) or with the suitable tool (not included) on the brake wire (picture 2)

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• We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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